

# ARIES<sup>®</sup>

ARIES<sup>®</sup> Systems

## ARIES<sup>®</sup> Group A Strep Assay



The ARIES<sup>®</sup> Group A Strep Assay is a real-time polymerase chain reaction (PCR) based qualitative in vitro diagnostic test for the direct detection of *Streptococcus pyogenes* (Group A β-hemolytic *Streptococcus*) in throat swab specimens from patients with signs and symptoms of pharyngitis.

The ARIES<sup>®</sup> Group A Strep Assay offers an easy to use, automated system that delivers higher sensitivity and specificity than existing rapid tests, and a faster turnaround than culture methods.

- **Specific:** Identify patients with *Streptococcus pyogenes* infection to aid in the diagnosis of pharyngitis
- **Flexible and Efficient:** Run from 1 to 12 tests per batch, supporting both STAT testing and low to medium volume sample batching
- **Fully Integrated:** Automate all aspects of testing, from sample preparation through analysis
- **Fast Time to Results:** Answers in less than 2 hours enable timely treatment decisions
- **Error-reducing Safeguards:** Internal barcode scanning matches samples to cassettes and may reduce data input errors

### Performance

ARIES<sup>®</sup> Group A Strep Assay Performance Compared to Bacterial Culture followed by Identification with MALDI-TOF MS

Refer to Package Insert for additional details: Luminex Corporation | ARIES<sup>®</sup> Group A Strep Assay (IVD) Kit Package Insert.


ARIES <sup>®</sup> Group A Strep Assay	Bacterial Culture		
	Positive	Negative	Total
<b>Positive</b>	156	10 <sup>2</sup>	166
<b>Negative</b>	4 <sup>1</sup>	448	452
<b>Total</b>	160	458	618 <sup>3</sup>
<b>95% CI</b>			
<b>Sensitivity</b>	97.5%	93.7% - 99.0%	
<b>Specificity</b>	97.8%	96.0% - 98.8%	
<b>PPV</b>	94.0%	89.3% - 96.7%	
<b>NPV</b>	99.1%	97.7% - 99.7%	

<sup>1</sup> Two (2) of the ARIES<sup>®</sup> Group A Strep Assay negative specimens that were positive by bacterial culture followed by identification with MALDI-TOF MS (i.e., False Negative) were Group A Strep negative by bi-directional sequencing analysis using analytically validated primers that targeted genomic regions distinct from the ARIES<sup>®</sup> Group A Strep Assay.

<sup>2</sup> Seven (7) of the ARIES<sup>®</sup> Group A Strep Assay positive specimens that were negative by bacterial culture followed by identification with MALDI-TOF MS (i.e., False Positive) were positive by bi-directional sequencing analysis using analytically validated primers that targeted genomic regions distinct from the ARIES<sup>®</sup> Group A Strep Assay.

<sup>3</sup> Five (5) specimens generated inconclusive results by comparator culture method (MALDI-TOF MS log(score) <2.00). All five specimens were excluded from the device performance calculations.

## Workflow

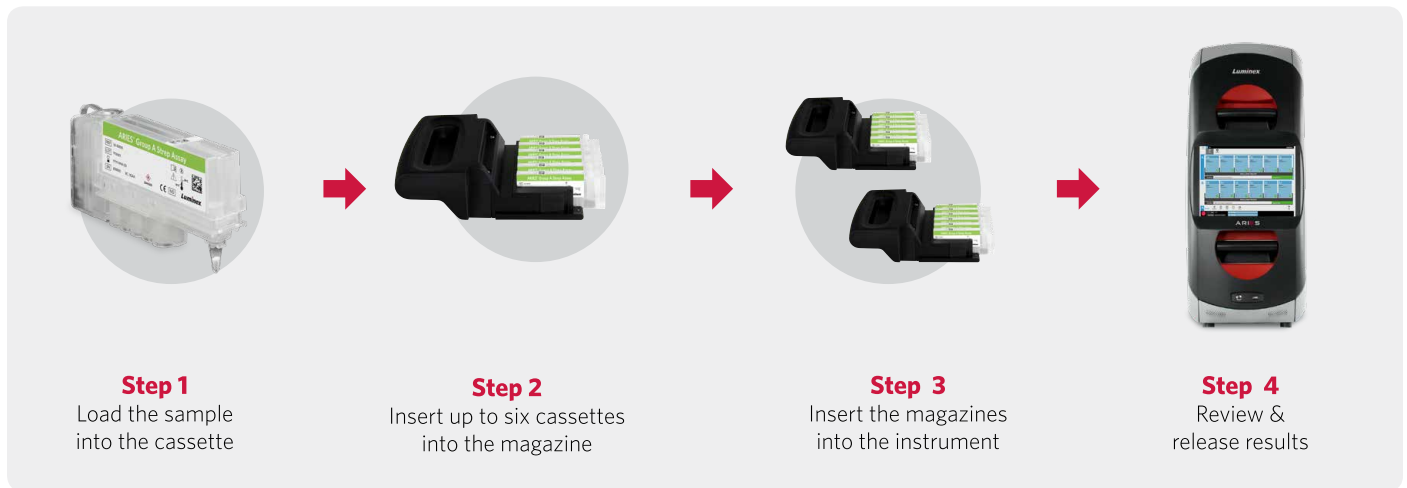

The operator simply adds specimen to the sample chamber, puts the cassette in the magazine, loads the magazine into the ARIES® System, and the run will start automatically.

## Usage and Targets

The ARIES® Group A Strep Assay can be used as an aid in the diagnosis of Group A Streptococcal pharyngitis. The assay is not intended to monitor treatment for Group A *Streptococcus* infections.

The ARIES® Group A Strep Assay is indicated for use with ARIES® Systems.

## Ordering Information\*

Product Name	Part Number
ARIES® Group A Strep Assay	50-10041 (24 tests)
ARIES® Group A Strep Assay Protocol File Kit	CN-0385-01 (one time order only)
ARIES® Two Module System Includes: Instrument System Operation Manual Two Magazines	Quick Guide Two Sample Prep Trays Power Cord Handheld Barcode Scanner and Stand ARIES-M12V1-IVD
ARIES® M1 System Includes: Instrument System Operation Manual One Magazine	Quick Guide Two Sample Prep Trays Power Cord Handheld Barcode Scanner and Stand ARIES-M6V1-IVD
SYNCT™ Software	CN-SW47

\* Products are CE Marked and FDA Cleared for IVD Use.

**Luminex®**  
complexity simplified.

**orders@luminexcorp.com or support@luminexcorp.com**

For In Vitro Diagnostic Use. Products are region specific and may not be approved in some countries/regions. Please contact Luminex at support@luminexcorp.com to obtain the appropriate product information for your country of residence. ARIES® Systems are class 1(I) laser products. Validation of the LIS compatibility must be performed by the end user.

©2017-2019 Luminex Corporation. All rights reserved. Luminex and ARIES are trademarks of Luminex Corporation, registered in the U.S. and other countries. SYNCT is a trademark of Luminex Corporation.

[www.luminexcorp.com/aries-gas](http://www.luminexcorp.com/aries-gas)

HEADQUARTERS

UNITED STATES

+1.512.219.8020

info@luminexcorp.com

EUROPE

+31.73.800.1900

europe@luminexcorp.com

CANADA

+1.416.593.4323

info@luminexcorp.com

CHINA

+86.21.8036.9888

info@luminexcorp.com

JAPAN

+81.3.5545.7440

infojp@luminexcorp.com